






BIOLUX_{PRO}

MAX WITH TENS

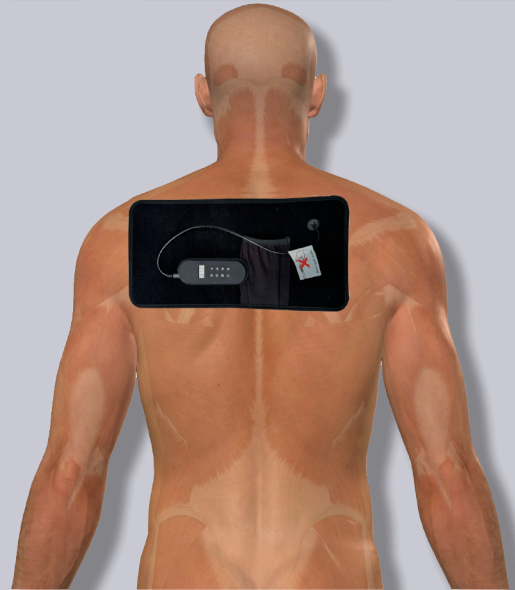

Burns, skin irritation, mosquito bites

1. Place BioluxPro on the affected area. You can do this while lying down.
2. In none of these cases, electrodes are necessary.
3. Press 
4. Press  select 660.
5. Press  select 10 mins.
6. Press  select L09.
7. If you feel too much heat during the process, press F at 40Hz. to pulse the light.
8. Repeat this process up to three times a day if necessary.



Proper hydration enhances results



This does not suggest or replace any medical treatment

